


- Don the N95 particulate respirator in a clean place.
- This User Guide describes how to don the respirator. Make sure to read the user instructions packaged with the respirator prior to use for proper usage.

Hi-Luck 350 Instructions how to don

①

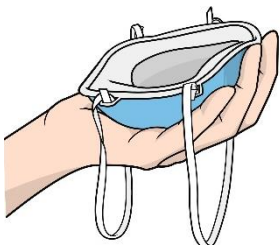


Place the respirator on your hand so that both headbands hang toward the back side of your hand and the notch of the facepiece lip faces toward the fingertip.

②



Place the respirator over your face so that the root of your nose is covered by the notch of the facepiece lip.

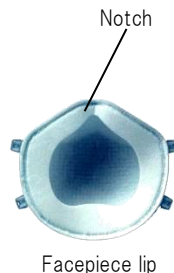

③



Pull the lower headband toward the back of your neck.

④



Pull the upper headband toward the back of your head.

⑤



Adjust the respirator position by shaking gently the respirator from side to side and up and down so that it snugly fits on your face.

Adjust the length of the headbands, as needed, following the procedure "Adjustment of headband." When donning is completed, perform user seal check.

●User Seal Check (Positive pressure method)



A user seal check is performed to check for air leakage between the respirator and the face to ensure that the respirator is properly donned. (A user seal check must be performed each and every time a respirator is donned.)

Cover the filter surface with your hands and exhale. Good fit is obtained if no leakage of air is observed between the respirator and your face.

If leakage of air is perceived, either adjust the position of the respirator on your face or adjust the length of the headbands.


●Adjustment of headband

To tighten



Pull the end of the headband while donning the respirator.

To loosen

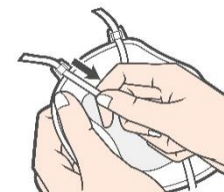

Do off the respirator and pull the headband as shown on the illustration.

- Doff the N95 particulate respirator in a clean place.
- This User Guide describes how to doff the respirator. Make sure to read the user instructions packaged with the respirator prior to use for proper usage.

Hi-Luck 350 Instructions how to doff

Remove the lower headband

①



Lift the lower headband with your fingers toward the above.

②



Remove the lower headband which is lifted over the top of your head.



Warning

- Doff the respirator without touching on the surface of respirator.

There are some potential risks of secondary infection if virus, bacteria, etc. adhere on the filter surface.


Remove the upper headband / Dispose of the respirator without touching on the filter surface

③



Lift the upper headband with your fingers toward the above.

④

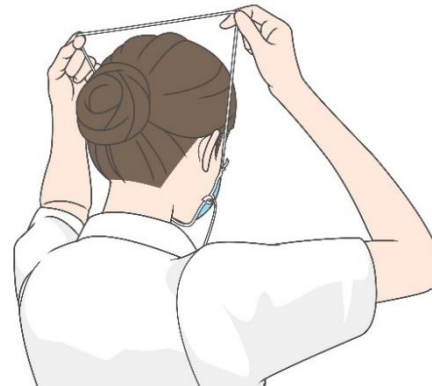

Remove the upper headband which is lifted over the top of your head.

⑤



After the respirator is pulled away from your face, dispose of the respirator without touching on the filter surface.